

IMPROVE SCARS

SCAR REDUCTION GEL

INHIBIT HYPERTROPHIC SCARS

Scientific formula, Gentle and skin-friendly

No cover, more breathable



- Traumatic scar
- Burn scar
- Surgical scar



Scar Reduction Gel

Manufacturer: Weihai Medison Medical Equipment Co.,Ltd

The Registered Address: Qujiahe Industrial Park, Yangting Town,
Huancai District, Weihai City, Shandong Province, China

Production License Number: Lu FDA Production License 20130088

Medical Equipment Registration Number:
Lu Machinery Registration Approval 20212140526



Product Information

【Product Name】 Scar Reduction Gel

【Product Model】 MDS-QBNJ01

【Specification】 10g、20g

【Product composition and structure】 The main component of the product is a dimethicone-vinyl dimethicone crosspolymer. This product is packaged in a tube or squeeze bottle. Products are provided in non-sterile form.



Product Main Performance

- The appearance of this product is transparent and colorless gel, with uniform texture and clean without impurities.
- Cold resistance: - 15 ° C, 24 hours, no oil-water separation in the paste after recovery to room temperature.
- Heat resistance: 40 ° C, 24 hours, no oil-water separation in the paste after recovery to room temperature.
- Viscosity: the viscosity shall not be less than 1000MPa·s

Scope of application:



This product is indicated for the treatment of hypertrophic and keloid scars (such as those resulting from general surgery, trauma and burns) on healed wounds and undamaged skin. Helps with scar improvement. This product is not suitable for old scars.

Instructions for use:



- Clean and dry the scar area.
- Take an appropriate amount of this product and apply it on the scar, and use your finger pulp to massage the scar repeatedly until it is dry.
- Twice a day, morning and evening.
- Please use this product under the guidance of professionals.


Contraindication:



- Don't use on unhealed wounds and damaged skin.
- It is prohibited for those who are allergic to product ingredients.

Applicable Department

Thyroid, obstetrics and gynecology, plastic surgery, pediatrics, surgery and other general departments.

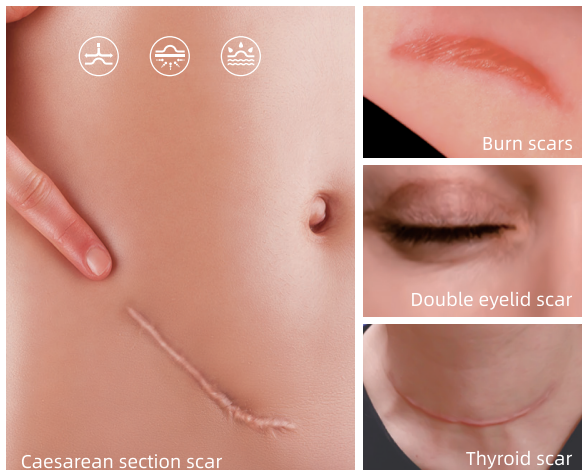

Precautions

- Avoid direct contact with eyes, mucous membranes, third-degree burns and unhealed wounds. In case of accidental contact, rinse immediately with plenty of water.
- For external use only.
- Not applicable for unhealed wounds.
- Do not use in the same place with external antibiotics or other external skin drugs.
- If the package is damaged, it is prohibited to use.
- Expired products are prohibited.
- In case of side effects such as redness, pain and irritation, please stop immediately. Consult your doctor if side effects occur.
- After the product is opened and exposed, please use it within 6 months.